

# LIFTKAR PTR

POWERED TRACKED STAIRCLIMBER



INSTRUCTION MANUAL

English

**SANO**   
makes life easier.

<b>INTRODUCTION</b>	<b>3</b>
<b>1 GENERAL SAFETY GUIDELINES</b>	<b>4</b>
<b>2 PRODUCT DESCRIPTION</b>	<b>8</b>
2.1 Controls - Handle Unit	9
2.2 Controls - Base Unit	10
2.3 LIFTKAR PTR – INDIVIDUAL COMPONENTS	10
2.4 TECHNICAL DATA	11
2.5 Indicators and Controls	11
2.6 Main Dimensions	12
<b>3 START-UP</b>	<b>13</b>
First Steps	13
3.1 Charge Battery	13
3.2 Inserting / Locking the Handle Unit to the Base Unit	13
3.3 Locking the Handle Unit	14
3.4 Detaching the Handle Unit	14
3.5 Removing the Handle Unit	14
<b>4 OPERATION</b>	<b>15</b>
4.1 Attaching the Wheel Chair	15
4.2 Operation on the Stairs	16
4.3 Removing the Wheel Chair	18
4.4 Using the Tracked Stairclimber on Level Surfaces	18
4.5 Tilting Angle Sensor	19
4.6 Battery Status Warning	19
4.7 Recommendation for Charging the Battery Pack	20
4.8 Safety Release System	20
<b>5 CLEANING AND DISPOSAL</b>	<b>21</b>
5.1 Cleaning and Disinfection	21
5.2 Disposal	22
5.3 Re-Use / Second-Hand Use	22
5.4 Fire-Fighting Measures	22
5.5 Safety Check	23
5.6 Recalls and corrective actions	23
<b>6 WARRANTY AND COSTUMER SERVICE</b>	<b>24</b>
6.1 Modification of the Tracked Stairclimber	24
<b>7 WARRANTY AND LIABILITY</b>	<b>24</b>
7.1 Warranty	24
7.2 Liability	25
<b>8 DECLARATION OF CONFORMITY</b>	<b>26</b>
<b>NOTES</b>	<b>27</b>
<b>CONTACT</b>	<b>28</b>

# INTRODUCTION

## CONGRATULATIONS!

**WITH LIFTKAR PTR YOU RECEIVE A SAFE TRACKED STAIRCLIMBER. THE PURPOSE IS THAT IT HELPS YOU SAFELY CONVEY WHEELCHAIR USERS UP AND DOWN STAIRS.**

The LIFTKAR PTR is a mobile unit that has been especially developed for operating on straight steps. Wheelchair users can now move around more freely and helpers no longer need to strain their backs on the stairs.

The LIFTKAR PTR can quickly be dismantled into two handy parts by shifting two levers: the base unit including rechargeable battery weighs approx. 28 kg, the handle unit approx. 11 kg. The LIFTKAR PTR ensures safe handling on the stairs. Regardless of whether you operate a professional mobility service or are a friend, relation or partner, we wish you much satisfaction in using this unit. This instruction manual is designed to help you learn how to use the LIFTKAR PTR.

In order to update you about product news as well as technical changes (e.g. options, accessories) we ask you to register your product online.

<https://www.sano-stairclimbers.com/product-registration>



THE INSTRUCTION MANUAL DOES NOT REPLACE THE OBLIGATORY TRAINING PROVIDED BY AN AUTHORISED SPECIALIST.




LIFTKAR® PTR (incl. options headrest, backrest cushion and foldable wheelchair brackets)





For wheelchairs with brake levers at the push handles it is obligatory to use mini ramps.

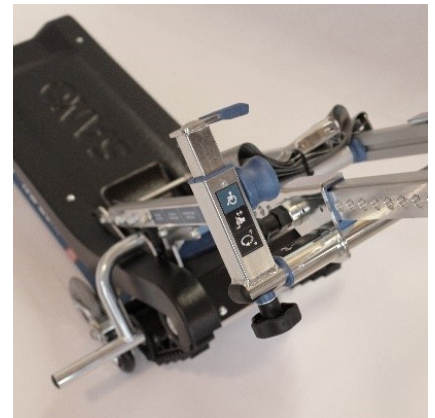

## 1 GENERAL SAFETY GUIDELINES



### **CAUTION - IMMINENT DANGER OR HARMFUL SITUATION POSSIBLE**

If not avoided, physical injury or damage to property will result.



### **WARNING - IMMINENT DANGER POSSIBLE**

If not avoided, death or severe physical injury can result.



### **DANGER - IMMINENT DANGER**

If not avoided, death and severe physical injury will result.



#### **HERSTELLER**

Manufacturer



#### **HERSTELLUNGSDATUM**

Date of manufacture



#### **SERIENNUMMER**

Serial number



#### **TROCKEN AUFBEWAHREN**

Keep dry



#### **MEDIZINPRODUKT**

Medical device



#### **CE-KENNZEICHNUNG**

CE-Marking

**SERBISCHE CE-KENNZEICHEN**

Serbian CE-Marking

**ELEKTROSCHROTT**

Electronic Waste

**TEMPERATURGRENZWERTE  
ANWENDUNGSBEREICH 5 ° C BIS 40  
° C**

Temperature limit Range of application  
5 ° C till 40 ° C (41 ° F till 104 ° F)

**EINDEUTIGE  
PRODUKTIDENTIFIZIERUNG**

Unique product identification

**GEBRAUCHSANWEISUNG  
BEACHTEN**

Consult instructions for use

**WARNUNG VOR HANDVERLETZUNG  
DURCH RIEMENANTRIEB**

Warning of hand injury due to belt drive

**NOTE**

- Please read this instruction manual carefully. Follow all instructions in this manual and take notice of the warning signs and pictograms on the unit. Climbing stairs presents a risk in itself and not complying with these instructions could cause accidents.
- The tracked stairclimber may only be used for the purpose for which it has been designed – transporting people on straight stairs.
- The LIFTKAR-PTR must be operated by people who
  - have proof of having been trained in how to operate the tracked stairclimber
  - can walk upstairs backwards
  - are both physically and mentally sound
  - are able to maintain balance without great physical strain.
- The LIFTKAR PTR must NOT be operated by the following people:
  - People under the age of 16
  - People under the influence of medication, drugs or alcohol
  - Pregnant people
  - Children are not permitted to use the tracked stairclimber.
- If an accident should occur with the stairclimber, this must be reported to your specialist dealer immediately (vigilance!).
- The wheelchair must be in technically trouble-free condition (no damage to the frame etc., no missing or loose fittings, no modifications, tires and brakes in good condition). If in doubt, contact your specialist dealer before using
- The LIFTKAR PTR is not designed for travelling extended distances over horizontal surfaces.
- Do not use the LIFTKAR with a passenger until you have practiced operating it perfectly without a payload. Next, try operating it again with somebody as lightweight as possible who is not disabled. The test passenger should hold onto a handrail or a second person during the climb.
- Use on wet, slippery surfaces is not recommended.



## CAUTION

- Do not use the LIFTKAR with a passenger until you have practiced operating it perfectly without a payload. Next, try operating it again with somebody as lightweight as possible who is not disabled. The test passenger should hold onto a handrail or a second person during the climb.
- Never reach into the stairclimber tracks (risk of trapping/squashing)!
- To transport the device itself, we recommend dismantling the LIFTKAR PTR.
- The LIFTKAR PTR has a powerful 12Ah/24Volt rechargeable battery that should always be fully-charged before using the tracked stairclimber. If, for whatever reason, the battery pack should run out of power while in operation on stairs, immediately move the stairclimber down the stairs and fully recharge the battery. It is always possible to move down at least one flight of stairs, even with a rechargeable battery that is almost empty. The first sign of the battery pack running out is indicated by the stairclimber's performance. The LIFTKAR PTR becomes slower and appears to be struggling. Inexperienced operators are warned by the LED display: **if the battery pack needs to be recharged the LED display starts flashing, alternating between red and green and there is an intermittent acoustic warning signal.**
- The following rules apply regarding ambient conditions for storing and operating the LIFTKAR tracked stairclimber: avoid exposing the tracked stairclimber to high temperatures (above 40°C) such as direct sunlight or high air humidity (above 70%) as a consequence of using in a sauna area or in the rain, or similar, due to the risk of damaging the stairclimber as a result of overheating. Likewise, avoid exposing the tracked stairclimber to very low temperatures below 5°C. Do not use in the vicinity of sensitive medical equipment.



## WARNING

- For transporting people using LIFTKAR PTR tracked stairclimbers, only use wheelchairs fitted with a safety belt or other personnel retention system.
- It is essential that passengers who are not able to sit independently are strapped in.
- Do not use the tracked stairclimber if unusual noises and/or vibrations occur while the LIFTKAR stairclimber is in operation. Withdraw the stairclimber from use and have it inspected by an authorized specialist.
- Never use the LIFTKAR PTR on stairs at an inclination of more than 35°.
- Make sure that the stairs to be climbed are designed to bear the combined load of tracked stairclimber + wheelchair + weight of persons.
- Never operate the stairclimber on heavily damaged stairs (e.g. if the edges of the steps are worn away).
- It is not possible to use the tracked stairclimber on spiral staircases or stairs with tight corners.
- To prevent yourself from slipping off steps, make sure you are wearing non-slip footwear.
- To avoid accidents, do not use the stairclimber on wet, slippery or waxed stairs. Carpet coverings must also be non-slip.  
The person being transported must not hang their arms to the side or move their arms suddenly. Instead, their arms should rest on the armrests or their thighs.



## DANGER


- Never operate the LIFTKAR PTR on surfaces that are wet, slippery, polished or waxed, smooth, icy or generally do not give the operator sufficient stability, and impair the adhesion of the stairclimber tracks. Rugs and carpets can also present a risk.
- Under no circumstances may you let go of the handle while operating the stairclimber on the stairs. Always operate the stairclimber with both hands (Danger of falling).
- Make sure that nobody is below the unit while in operation on stairs (Danger of collision).
- **Make sure you always wear closed, non-slip shoes when operating the LIFTKAR (Danger of falling).**
- Without exception, the stair climber must not be used in explosion-proof areas.
- Never overload the device. The maximum load capacity (model-dependent) of the LIFTKAR PTR is 130 kg or 160 kg (weight of person + weight of wheelchair). Make sure that your wheelchair (!) is designed to bear the weight of the person to be transported.
- When driving down the stairs, stop the drive in the direction of the step edge as soon as the "STOP" marker on the side is level with the step edge. (Danger of falling!)

## 2 PRODUCT DESCRIPTION



- |                               |                                     |
|-------------------------------|-------------------------------------|
| 1. Headrest *                 | 6. Backrest Cushion *               |
| 2. Wheel Chair Retention Clip | 7. Retention Bolt                   |
| 3. Handle Unit                | 8. Retention Lever                  |
| 4. Base Unit                  | 9. Controls - Base Unit             |
| 5. Controls - Handle Unit     | 10. Foldable Wheel Chair Brackets * |

\* optional

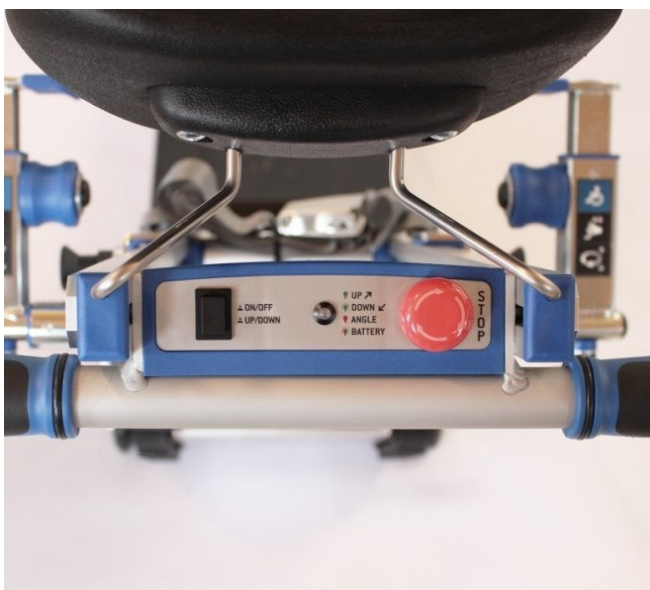


Controls - Handle Unit



Controls - Base Unit



## 2.1 CONTROLS - HANDLE UNIT

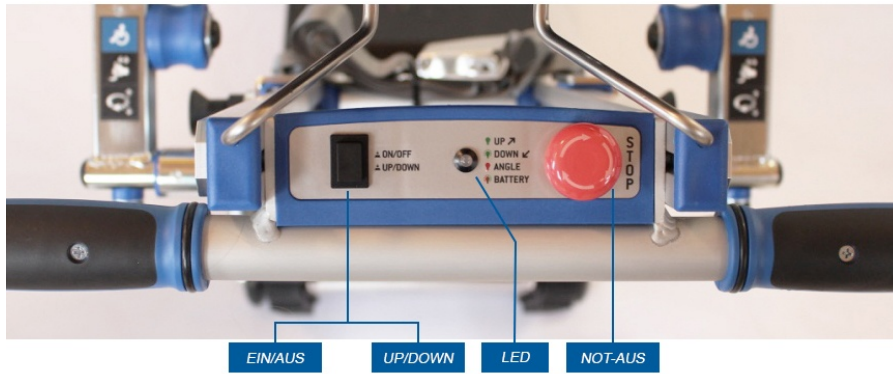

### ON/OFF - UP/DOWN

- If the LIFTKAR PTR is switched off, switch it on by pressing this button. The LIFTKAR PTR automatically switches on in UP mode.
- The LED **illuminates green**.
- Press this button to switch to DOWN mode. LED **flashes green**.
- Press this button longer (approx. 2 seconds) to switch off the LIFTKAR PTR. After approx. 5 minutes of inactivity, the unit switches off automatically to save battery power.

### LED

- **Illuminates green** in UP mode.
- **Flashes green** in DOWN mode.
- **Changes to red** if the stairs are too steep (36° - 40°). The speed of the LIFTKAR PTR is reduced automatically.
- **A red light and an acoustic warning go on** when the unit is overloaded, or the stairs are too steep. PTR stops.
- **Flashing red and green and an acoustic warning go on** if battery power is low.


### EMERGENCY STOP

- Stops the LIFTKAR PTR tracked stairclimber.
- Reset the Emergency STOP by pulling or rotating the switch.

## 2.2 CONTROLS - BASE UNIT

### KEY

The key protects the LIFTKAR PTR against unauthorised use.

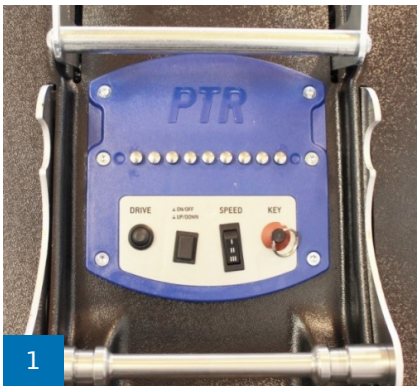

### ON/OFF

Same function as on handle unit.



### WARNING

Use only for loading purposes (e.g. stowing in car).

### DRIVE

Starts the drive chain in the chosen climbing mode.

### SPEED

Preselect switch for stair climbing speed

Speed I ..... 5,5m/min  
Speed II ..... 6,5m/min  
Speed III ..... 7,5m/min

## 2.3 LIFTKAR PTR - INDIVIDUAL COMPONENTS



LIFTKAR PTR – COMPONENTS (SCOPE OF DELIVERY)

## 2.4 TECHNICAL DATA

Maximum load (incl. wheel chair)	130 kg / 160 kg
Minimum wheel chair width (frame inside)	420 mm
Max. inclination	35°
Total weight	39 kg
Weight of base unit (incl. batteries)	28 kg
Range	up to 1000 steps *
Minimum landing space	970 x 970 mm
Sound pressure level	below 80dB
Required pressure force for the controls:	max. 5 N

\* depending on overall weight to be transported

---

## 2.5 INDICATORS AND CONTROLS


### BASE UNIT

- Key switch
- ON/OFF - UP/DOWN switch
- Speed selector switch I / II / III
- DC charging socket
- Main fuse


### HANDLE UNIT

- ON/OFF - UP/DOWN switch
  - Multi-function LED – battery status and inclination angle
  - EMERGENCY STOP
  - Activation button
-

## 2.6 MAIN DIMENSIONS

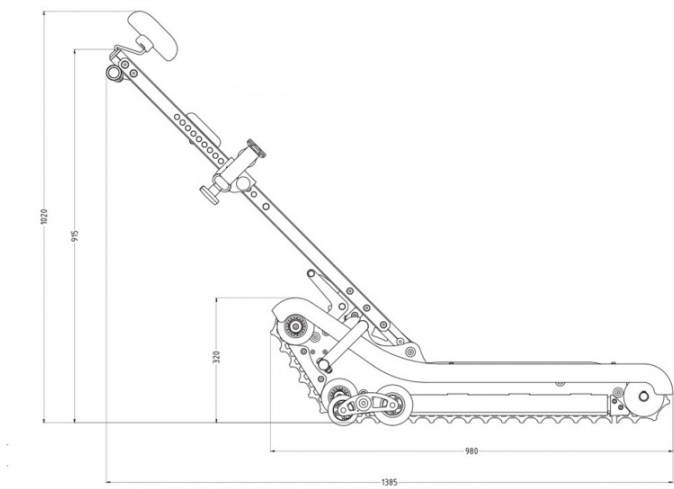

SIDE VIEW

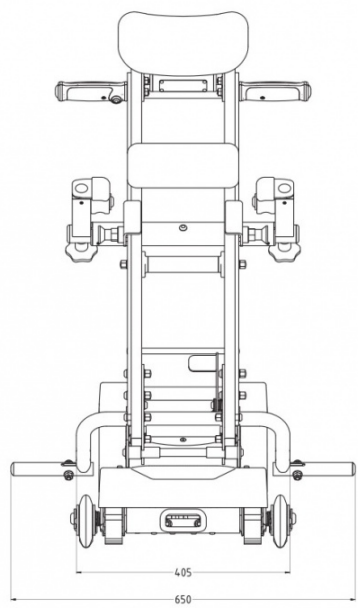

FRONT VIEW



# 3 START-UP

## FIRST STEPS

---

### 3.1 CHARGE BATTERY

Fully charge the 24V/12Ah battery pack installed in the base unit. This can take **up to 6 hours** depending on the residual charge. The charging socket is located on the back of the base unit ([figure 1a](#)).



Liftkar PTR is equipped with an inbuilt battery charger. During charging process a LED ([figure 1b](#)) which you can find in the left side wall of the device chassis indicates the charging level:

**Orange:** charging process running


**Green:** charging process terminated, fully charged batteries



### 3.2 INSERTING / LOCKING THE HANDLE UNIT TO THE BASE UNIT




Place the U-shaped yoke on the underside of the handle unit on the round support bar in the base unit ([figure 2a](#)).

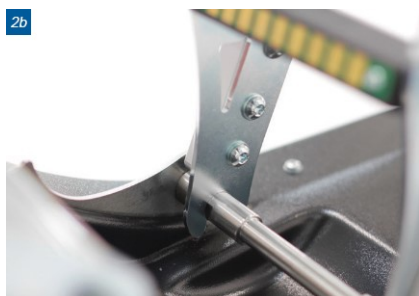

Make sure that the yoke engages with the grooves of the support bar ([figure 2b](#)). By pivoting the handle unit for- / backwards the engagement is facilitated.

### 3.3 LOCKING THE HANDLE UNIT

Pivot the handle unit into the operating position. The spring-loaded retention lever engages automatically. Additionally, the safety lug holds the retention lever in position (figure 3).

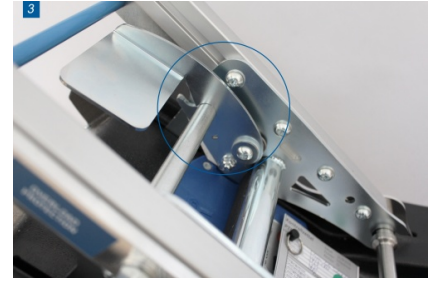

#### WARNING

Before operating the tracked stairclimber **always** control the position of the retention lever. The safety lug **must** be engaged!

### 3.4 DETACHING THE HANDLE UNIT



Press the safety lug forward (figure 4a), subsequently draw the retention lever backwards into its lower limit position (figure 4b).

### 3.5 REMOVING THE HANDLE UNIT



Pivot the handle unit forward and pull slightly up-wards. Once the snap-in position is reached the handle releases itself automatically.

## 4 OPERATION

### 4.1 ATTACHING THE WHEEL CHAIR

#### 4.1.1 PLACE WHEEL CHAIR ON STAIRCLIMBER

Roll the wheelchair along the centreline of the base unit until the tyres of the wheelchair wheels touch the wheelchair support.



#### 4.1.2 PIVOT HANDLE UNIT TOWARDS WHEEL CHAIR

Release handle unit and pivot it towards the wheelchair until the handle unit is approximately vertical (parallel to the wheelchair backrest).

#### 4.1.3 ADJUST POSITION OF WHEEL CHAIR RETENTION BRACKETS

Position the wheelchair retention bracket directly underneath the bend in the wheelchair handle.

By pulling on both locking pins you can adjust the vertical position of the retention bracket to the height of the wheelchair.



#### 4.1.4 SECURING THE WHEEL CHAIR

Adjust the position of the two slide blocks (to the left and right of the bracket) and fix them in place by tightening the two hand bolts.

Adjust the headrest and pivot the handle unit back into the operating position.

Keep both your hands on the handle unit while you do this. The spring-loaded retention lever automatically moves into its front limit position.



#### NOTE

- Make sure that the safety lug has engaged properly.
- Apply the brake on the wheel chair.
- Make sure the power is switched off before getting on or off the stairclimber to make sure it does not start up accidentally.



#### WARNING

Never leave the person to be conveyed in this position unattended.

## 4.2 OPERATION ON THE STAIRS

### 4.2.1 SWITCHING ON

Switch on the tracked stairclimber using the key switch and the ON/OFF button.

The tracked stairclimber is automatically set to move UP. The LED is illuminated permanently green.



### 4.2.2 MOVING UP

Position the tracked stairclimber (figure 2a) so that it is at an angle of 90° to the stairs.



Always hold the handle unit securely with both hands. Use the direction selector switch on the handle unit display to select “UP” (figure 2b). The LED illuminates permanently green.



Stand behind the handle unit of the tracked stair-climber as instructed during training by authorised personnel. Press the activation button on the right handle (figure 2c).





#### WARNING

- Never move on stairs with an inclination of more than 35° and during operation the handle unit must always be held with both hands.
- Always move on stairs at an angle of 90° (length of tracked stairclimber to steps).
- The integrated angle sensor (angle LED) is designed merely as an indicator and does not represent a safety feature.

### 4.2.3 REACHING THE TOP STEP

Stop the tracked stairclimber as soon as the „STOP“- marks on the side have reached the edge of the top step (figure 3).

Now gradually tilt the tracked stairclimber back until the tracks rest on the floor of the landing. Press the activation button again. The tracked stairclimber then finishes moving onto the landing.



### 4.2.4 MOVING DOWN

Position the tracked stairclimber at an angle of 90° to the stairs so that the passenger is facing the stairs. Use the direction selector switch on the handle unit display to select “DOWN” (figure 4). LED flashes green.

Hold the handle unit securely with both hands and press the activation button to move the tracked stairclimber towards the stairs until the “STOP” mark on the side reaches the edge of the top step. Stop the tracked stairclimber and, using both hands, tilt the handle unit forward until the stairclimber tracks rest on the steps. Then press the activation button to continue moving down the stairs.



#### WARNING

Before operating the tracked stairclimber it must be at an angle of 90° to the stairs.



#### DANGER

When driving down the stairs, stop the drive in the direction of the step edge as soon as the “STOP” marker on the side is level with the step edge. (Danger of falling!)

### 4.2.5 SAFETY CHECK

Although the tracked stairclimber is equipped with an angle warning LED, we recommend checking the precise inclination of the stairs before climbing them for the first time.



## 4.3 REMOVING THE WHEEL CHAIR

### 4.3.1 RELEASING

Release the safety lug (figure 1a) and move the retention lever into the rear position (figure 1b).

Pivot the handle unit carefully towards the wheelchair until the wheelchair reaches the ground and the handle unit is in a vertical position. Then undo the manual bolts and pull both retention blocks to the side out of the wheelchair backrest.



#### NOTE

Perform this procedure only on horizontal ground, well away from the stairs.

## 4.4 USING THE TRACKED STAIRCLIMBER ON LEVEL SURFACES

### 4.4.1 MOVING ON LEVEL SURFACES

In order to move the wheelchair on the tracked stairclimber or to turn the combination on landings, push the handle unit slightly downwards (figure 1).

This raises the stairclimber tracks so that the tracked stairclimber can be shifted on the rear left-hand and right-hand side wheel units.



## 4.4.2 BASE UNIT FUNCTIONS

The following controls are located on the rear side of the base unit:

- **Combined ON/OFF, UP/DOWN switch**
- **Key switch**
- **Speed selector switch**

It is possible to operate the base unit without the handle unit. This mode of operation is provided exclusively for loading the base unit and for service activities.

The base unit is operated using the ON/OFF button. After switching on, the base unit is automatically set to Move UP mode. Pressing the ON/OFF button again switches the unit to the Move DOWN mode. Pressing the ON/OFF button for a longer period switches the unit off.

Use the speed selector switch to choose between one of three speed settings.

---

## 4.5 TILTING ANGLE SENSOR

Your LIFTKAR PTR is equipped with a tilting angle sensor, which detects the inclination of the LIFTKAR PTR on stairs. In the case that the stair angle exceeds 35°, the device clearly reduces its driving speed and the LED on the handle unit illuminates permanently red. In the event of an extreme stair angle, the unit stops climbing upstairs. (A warning tone as well as the red illuminated LED signalizes this condition.)

It is still possible to move downstairs in creep speed in order to bring the PTR out of the danger zone onto the next landing.

- Never move on stairs with an inclination of more than 35°.
  - The integrated tilting sensor does not substitute a check of the stair angle by the operator before the first movement.
- 

## 4.6 BATTERY STATUS WARNING

In case of low battery level, the LED on the handle unit starts flashing red/ green, as well as an alternating warning tone alerts the operator. It is still possible to drive 3-4 levels after the first alert.

---

## 4.7 RECOMMENDATION FOR CHARGING THE BATTERY PACK

The lead gel cells inside the battery pack are maintenance-free, gas-tight and rechargeable. Their service life depends largely on the number of charging/discharging cycles. For example, it is possible to partially discharge lead batteries 1000 times, drawing more than 200 times the full capacity of the battery, providing the battery is never fully discharged.

- Avoid discharging the battery completely. Charge the battery pack as often as possible.
- Lead batteries are susceptible to self-discharging. The battery pack also needs to be looked after in that it must always be fully-charged. Completely discharging the battery will shorten its service life. Lead-gel cells have a long service life if they are regularly fully charged as soon after use as possible. That is why the battery pack should be connected up to the power connector cord after each use.
- If the lead acid batteries have been damaged, they can be replaced at authorized SANO dealers. The old lead batteries are fully recyclable and are not hazardous waste. If the batteries should become damaged it is possible to have them replaced in a workshop of authorized SANO partners. The old lead batteries are fully recyclable and are not to be disposed of.
- The ideal temperature for charging is between 20 and 25° C. Temperatures that are too cold or too hot will affect the battery's capacity.



### NOTE

If the battery pack is not fully charged, or it loses its charge suddenly, not only will the speed of the Liftkar PT be slower, but its capacity is reduced as well. This means that overload mode may be reached even with a relatively light load.

## 4.8 SAFETY RELEASE SYSTEM

The safety release system is positioned on the left side of the base unit (figure 1). In the event of an unpredictable machine break down (e.g. technical defect, power supply failure), this system allows the operator to manually move down the tracked stairclimber onto the next landing.

By pulling the safety release wire, the electromagnetic brake is disengaged manually. Due to its own weight (incl. wheelchair + weight of person) the LIFTKAR PTR starts moving downstairs.





**ATTENTION:**

- Before activating the safety release system always press the emergency stop button and remove the key.
  - Safety first - if safety release system is required, make sure that a second person (helper) is present, who activates the safety release system.
  - By pulling the safety release wire (helper) and pressing the handle unit downwards (operator), the device starts moving.
  - The device can be stopped anytime by releasing the safety release wire. Only move the stairclimber onto the next landing.
  - If the total weight (tracked stairclimber and passenger) is low, the operator will need to support travel down the stairs by pushing in the direction of travel.
  - After usage of the safety release system your LIFTKAR PTR must undergo a complete safety check by an authorised SANO partner.
- 

## 5 CLEANING AND DISPOSAL

### 5.1 CLEANING AND DISINFECTION

**Our motto is:**

A CLEAN MACHINE MOVES MORE SMOOTHLY.

- Cleaning with a standard household detergent is quite sufficient.
  - Never use a high-pressure cleaner or similar.
  - If required, the surfaces can be cleaned with a commercially available alcohol-based equipment disinfection agent (such as Sagrotan®).
-

---

## 5.2 DISPOSAL

The LIFTKAR PTR tracked stairclimber is a durable product. At the end of their useful life, the tracked stairclimber components and charger should be disposed of properly.

- The tracked stairclimber does not contain any hazardous materials and is fully recycling-compatible. The printed circuit boards and the rechargeable battery must be disposed of using a suitable recycling process.
- Do not dispose of the battery pack in household waste.
- Correct and comprehensive disposal of the tracked stairclimber can be processed and invoiced for by the manufacturer.



---

## 5.3 RE-USE / SECOND-HAND USE

- Before re-using the stairclimber or using it second-hand, make sure it undergoes a safety check and cleaning as per manufacturer's instructions by specialist personnel.
- All operators must have proof of having been trained in how to operate the stairclimber.

---

## 5.4 FIRE-FIGHTING MEASURES

- **Extinguishing agent:** Suitable: Dry chemical, sandy soil, carbon dioxide, suitable foam
  - **Firefighting:** Protective equipment: Wear self-contained breathing apparatus and protective clothing to avoid contact with skin and eyes.
  - **Specific hazards:** Toxic fumes released during fire conditions.
-

## 5.5 SAFETY CHECK



### NOTE

Safety inspections may only be carried out by the manufacturer, SANO service technicians or authorised dealers.



### NOTE

We recommend a safety check is carried out every two years. The inspection sticker indicates the date by which the next inspection should be carried out. A check is recommended especially after a change of user or if the stairclimber is going to be used again after a longer period.



**Safety checks are important to ensure that the device is working properly. The specialist dealers authorised by us can carry out this inspection on your device.**

## 5.6 RECALLS AND CORRECTIVE ACTIONS

We will inform you if new important safety information regarding the operation of your LIFTKAR PTR becomes available. The current version of this instruction manual can be found at [www.sano.at](http://www.sano.at). Should subsequent changes or corrective actions be needed to the product, we will inform you and/or our authorised dealers in writing of all the serial numbers affected. If your LIFTKAR PTR is affected by a product recall, you will be informed on what happens next. We recommend that you register your device on our website [www.sano-stairclimbers.com/product-registration](http://www.sano-stairclimbers.com/product-registration).

## 6 WARRANTY AND COSTUMER SERVICE

### 6.1 MODIFICATION OF THE TRACKED STAIRCLIMBER

- Modifications of any kind are not permitted and make void all liability and warranty claims to the manufacturer.
- Modifications to the tracked stairclimbers may only be made by the manufacturer.



#### OUR RECOMMENDATION!

In order to ensure an extended service life of the tracked stairclimber, we recommend having the tracked stairclimber serviced at regular intervals by an authorised SANO dealer. Please contact your SANO dealer for more information.



#### COSTUMER SERVICE

Costumer Service: +43 7239 / 510 10

## 7 WARRANTY AND LIABILITY

### 7.1 WARRANTY

The warranty period for the climbing unit and handle unit on all models of the LIFTKAR PTR tracked stairclimber is 24 months. Battery packs have a warranty period of 6 months. In both cases the warranty period starts on the day the product is handed over to the costumer.

The following are excluded from the warranty:

- parts subject to natural wear and tear
- damage that occurs from using the unit for a purpose it was not intended
- unauthorised modifications to the unit or accessories
- maintenance work required due to continuous operation
- faults occurring due to incorrect operation and/or failure to comply with the instruction manual, accidents, negligent or violent damage, damage due to fire and water, force majeure and other causes outside our control.



---

## 7.2 LIABILITY


SANO Transportgeraete GmbH is not liable as manufacturer for any damage, if:

- the LIFTKAR PTR is used for a purpose for which it is not intended.
- repairs, installation or other work have been carried out by unauthorised persons.
- the instructions in this manual are not complied with.
- non-SANO components are fitted or linked to the LIFTKAR PTR.
- original components are removed.



Please contact us for a list of authorised SANO service centers.

## 8 DECLARATION OF CONFORMITY

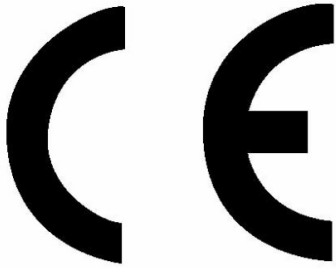

SANO Transportgeräte GmbH declares under its sole responsibility that the following LIFTKAR stair climber models comply with the essential requirements.

- the requirements of Regulation (EU) 2017/745 on medical devices (MDR) as a Class I medical device.
- Conformity assessment procedures in accordance with Annex IX (MDR).

Intended use: For transporting persons with walking disabilities up and down stairs

Classification: Class I, based on Annex VIII, Chapter III, Rule I, EU2017/745 (MDR)

Any changes to the product without our prior consent will invalidate this declaration.

### **Applied standards, in particular:**

ISO 7176-28:2012

ISO 7176-14:2008

ISO 7176-21:2009

This declaration of conformity is valid until one of the listed types is modified.

This declaration loses its validity if the product is modified without our consent.

A handwritten signature in black ink, appearing to read 'J. Bierma'.

Ing. Jochum Bierma, Managing Director

Artikel Daten für Deutschland / Article data for Germany		
Art.Nr.	Name	UDI / EAN Code
075801	LIFTKAR PTR 130 - HM	9120047773109
075802	LIFTKAR PTR 160 - HM	9120047773116
075803	LIFTKAR PTR-L 160 - HM	9120047773123
075805	LIFTKAR PTR-L 130 - HM	9120047773130
Artikel Daten für alle anderen Länder / Article data for all other countries		
Art.Nr.	Name	UDI / EAN Code
075701	LIFTKAR PTR 130	9120047773000
075702	LIFTKAR PTR 160	9120047773017
075703	LIFTKAR PTR-L 160	9120047773079
075705	LIFTKAR PTR-L 130	9120047773093

## NOTES

Model:	
Serial No.:	
Manufacturing date:	
Date of purchase:	
Dealer:	

# CONTACT

SANO Transportgeraete GmbH  
Gewerbezeile 15  
4040 Lichtenberg / Linz  
Austria

Phone +43 7239 / 510 10 100  
Fax +43 7239 / 510 10 001  
office@sano.at

[www.sano.at](http://www.sano.at)

SANO Deutschland GmbH  
Geigelsteinstraße 10  
83080 Oberaudorf  
Germany

Tel. +49 8033 / 308 96 0  
Fax +49 8033 / 308 96 17  
info@sano-treppensteiger.de

[www.sano-treppensteiger.de](http://www.sano-treppensteiger.de)

SANO UK Powered Stairclimbers Ltd.  
Bristol Court, Betts Avenue  
Martlesham Heath  
Ipswich, Suffolk / IP5 3RY, England

Tel. +44 1473 / 333 889  
Fax +44 1473 / 333 742  
info@sano-uk.com

[www.sano-uk.com](http://www.sano-uk.com)

Subject to changes without notice / LIFTKAR PTR 2025-12,

**[WWW.SANO-STAIRCLIMBERS.COM](http://WWW.SANO-STAIRCLIMBERS.COM)**

